


Endobronchial Valve Evaluation Workflow Quick Guide

Requirements


- SentrySuite™ 3.30 with Maintenance Package 5 or later
 - Standard configuration of settings in 'Administration Dashboard':
 - 'Endobronchial Valve Analysis' **must be enabled***
 - 'Custom Property Field' must be set (default = 'Property 30')
 - 'Analyze visit' must be enabled (analysis occurs automatically after measurement)
- *If not enabled, contact Jaeger Technical Support for assistance*

View List of Eligible Patients (written to user-defined Property Field)


1. Open SentrySuite™ 'Review'

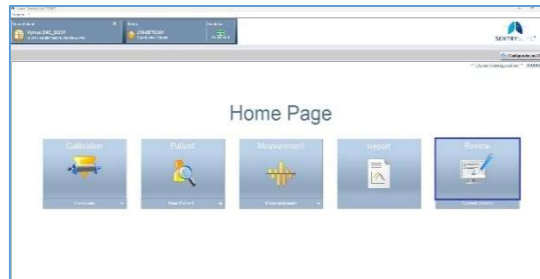

... or log in to 'Mobile Review'

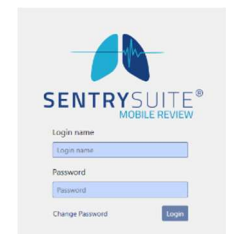


2. Click the '+' to **expand** the filters

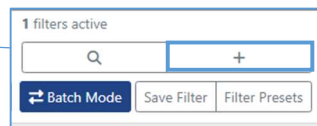

3. Select 'Yes' from the dropdown menu of 'Eligible for endobronchial valve evaluation'

4. Configure other applicable **filters** as desired (i.e., Date range, Location, Visit Status, etc.)


5. Click 'Q' (**Search**). The list of matched patients will be displayed with **endobronchial valve eligibility** in results

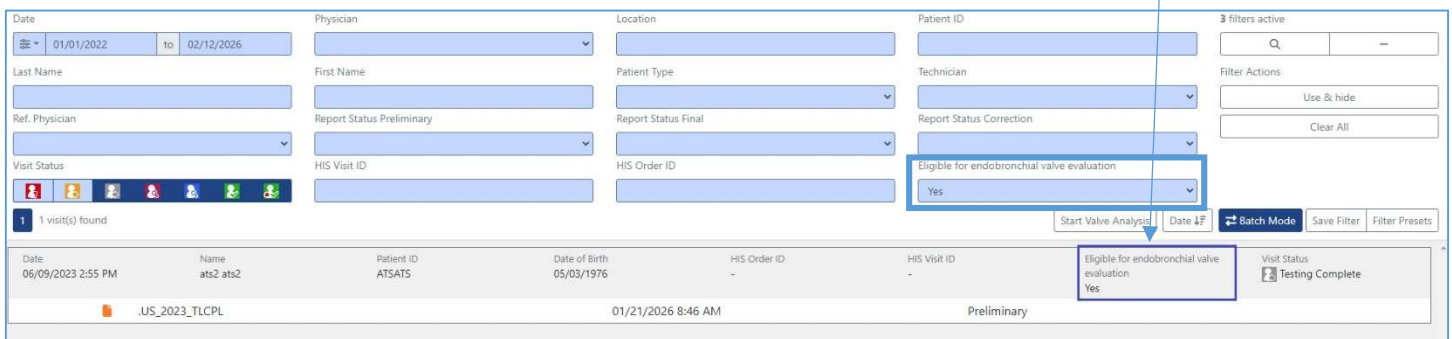

Re-running Endobronchial Valve Analysis (from 'Review' or 'Mobile Review')

NOTE: If Endobronchial Valve threshold criteria has been updated (i.e., FEV1, TLC and/or RV %Pred), re-run the analysis.

1. Select desired date range, or leave blank for all patients
2. Clear other filters (*only visits that match search filters will be analyzed*)
3. Click 'Start Valve Analysis' →
4. In window that appears, click 'Run' to begin analysis

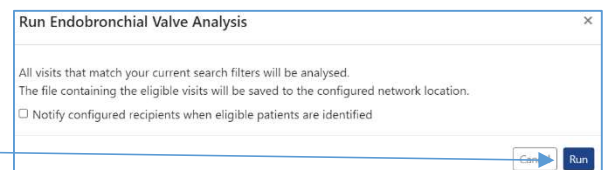

References

- SentrySuite™ Review user manual ("EN_SeS_Review.pdf")
- SentrySuite™ Mobile Review user manual ("EN_SeS_MobileReview.pdf")